

Which Wheel Lock Do You Need?

- Determine if lock style is bolt-through or clamp-on.
- Wheel locks for wheelchairs with fixed armrests do not have a spacer to offset the handle.
- Wheel locks for wheelchairs with detachable armrests have a spacer to offset the handle.

Fixed Armrests

Bolt-Through Style



No spacer to offset the handle

Clamp-On Style




Detachable Armrests

Bolt-Through Style



Extra space to offset the handle

Clamp-On Style

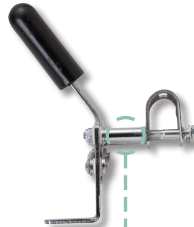

Any wheel lock adjustment should embed the wheel lock shoe at least 1/8" (3/16" for pneumatic tires) into the tire when engaged.