

UltraCare® XT leading bed evolution



Joerns® UltraCare® XT Bed System

The standard UltraCare XT has the same great features you enjoyed with last year's UltraCare bed plus newlydesigned controls, faster positioning speed, Advance Care and IPX4 Fluid Ingress Protection Rating to name a few advantages. Then, upgrade to suit your facility's needs. Choose some of the great options shown on the facing page including SafeLight, UltraLock, and UltraWide or experience other great features to customize your UltraCare XT bed. One bed. A total solution.

Features

- Toroidal architecture engineering of the bed's steel frame is up to 30 times stronger than angle iron
- 2 Therapeutic grid surface offers improved air flow and ultimate support for the sleep surface
- Integrated pendant connections on both sides of mattress support platform
- 4 Four-section mattress support platform offers full range of comfort and therapeutic positions, which may reduce skin shear
- 5 Attractive bed panel options available in a wide array of designs and finishes
- 6 Staff control allows caregivers flexibility to select which features may be adjustable by the resident

- 7 EasyLift Leg Lift provides quick positioning for improved circulation
- 8 Revolutionary arched legs deliver a super-strong, wide stance for equipment clearance
- Four DC motors supply amazingly quiet and smooth operation
- 10 7-inch to 30-inch travel range offers safety, whether resting at an ultra low height or providing care at the higher end
- Assist devices have superior functionality and perfect grip height
- 12 Sculpted button on assist device for easy, "press and go" operation
- UltraLock is available at the foot end or both the head and foot to maximize security or mobility at any height (option)
 - AdvanceCare features gravity-assist for easier and safer repositioning. Utilize Comfort Chair functionality for comfort, relaxation, improved circulation, and more ergonomic care by bringing the resident closer to the caregiver.



Simplified Controls

Ergonomically designed pendant meets geriatric needs featuring soft touch edges, finger grips and optional enhanced back lighting, raised buttons and illuminated lock-out status.

OPTIONS AND ACCESSORIES



UltraWide

Transform your UltraCare XT to the widest 500lb safe working load bed on the market. The extra width makes for safer repositioning and a more comfortable night's sleep. Choose from 39" or 42" extension kits.



UltraLock

Designed for mobility or security at any height, while allowing for a 7" low height. UltraLock provides quick, easy bed movement.



Trapeze and Trapeze Adapter

Trapeze adjusts to accommodate residents' reach; in addition, three working positions provide greater flexibility. Trapeze adapter allows attachment of a trapeze system.



Pendant Holder

Smooth design easily affixes to Joerns assist devices making pendant stowage and retrieval simple. Residents can even operate the pendant while sturdily stowed out of the way.

ASSISTS





Assist Device

Features a low profile design, avoiding feeling of restriction or restraint for the resident. Mounted at the perfect ergonomic height to assist residents in and out of bed. Soft-touch handle provides a sturdy handhold for improved resident grip.



Joerns PrevaMatt Foam Mattress

Our Deluxe and Elite mattresses with or without TEMPUR[®] material feature special mattress construction to keep residents in place when contouring the UltraCare XT bed.



Deluxe Assist Handle

New assist handle features an ergonomic and a sturdy soft touch handhold. The assist is mounted at an ergonomic height to best assist residents in and out of bed, and has an integrated pendant holder on each side.



Joerns Air System

Our complete line of therapeutic air systems include advanced functionality of CairRails and unique design to enhance your wound management program.

JOERNS® ULTRACARE® XT SPECIFICATIONS

Frame Construction

 All major structural components, including support surface platform, are designed with steel tubing in combination with patented toroidal construction to provide the utmost in structural stability

Additional Information

- All exposed steel surfaces are coated with epoxy powder paint
- Fasteners, screws, rivets and other non-painted parts are corrosion resistant
- All control switches are of momentary contact design
- Headpanel is easily lifted off the bed to allow for CPR in emergency situations

U.S. Patents

• 6473922, 6474869, 6519794, D436,474, 437,143, D445,614 Other Patents Pending

Certifications — ETL Listed

- USA: Conforms to UL 60601-1, 1st ED., 2003
- Canada: Certified to CAN/CSA C22.2 No. 601.1-M90, CAN/CSA STD C22.2 NO. 60601-2-38-03
- International: Conforms to IEC 60601-2-38, First ED., 1996, IEC 60601-2-38, Amendment 1, 1999

European: EN60601-1, EN60601-2-38

Mechanical Specifications

Product weight	220 lbs.
Maximum back angle	65°
Maximum knee angle	35°
Maximum height support surface platform above floor	30"
Minimum height support surface platform above floor	7"
 Mattress support platform translation - entire travel range 	1"
Mobility height without UltraLock	7"- 30"
Mobility height with UltraLock	7"- 30"

Electrical Specifications - North America

 Mains voltage 	120 VAC
Frequency	60 Hz
Input current	4.0 Amps
Control box output voltage	24V DC
Electrical protection	Class I or II, Type B
Duty cycle	10% (2 min. on/18 min. off)
Circuit protection	Auto reset current sensor

Size Specifications

Bed Size	Support Surface	Overall H Length ¹	Standard Overall HF Length ¹	AdvanceCare HF Length ²	Width ³	Low Height	High Height	Underbed Clearance⁴	Safe Work Load⁵
80"	35" x 80"	83-1/2"	85-1/2"	93.75"	35"	7"	30"	23-3/4"	500 lbs.
76"	35" x 76"	80"	82"	88.75"	35"	7"	30"	23-3/4"	500 lbs.

- 1 LENGTH: "H" is Headpanel only; "HF" is Head and Footpanel
- 2 LENGTH: Overall space required for AdvanceCare Positioning
- 3 WIDTH: With assist device brackets, 36-1/4"; with assist devices, 41"

4 CLEARANCE: Ample clearance between 14" and 30"

5 SAFE WORK LOAD: Includes bedding, resident, support surface, and all accessories

Finish

Joerns UltraCare XT comes in a soft tone finish.

We reserve the right to make dimension or specification changes, or deletions consistent with design trends, product improvements, or for reasons beyond our control.

Color options have been reproduced as faithfully as printing processes allow.

Environmental Conditions

Ambient Temperature	Relative Humidity	Atmospheric Pressure	Fluid Ingress Protection
+10° to +40°C	30% to 75%*	700hPa - 1060hPa	IPX4
-40° to +70°C	10% to 100%	500hPa - 1060hPa	
	+10° to +40°C	+10° to +40°C 30% to 75%*	+10° to +40°C 30% to 75%* 700hPa - 1060hPa

*NON-CONDENSING



Post Acute, Acute, HomeCare 2430 Whitehall Park Drive Charlotte, NC 28273 (P) 800.826.0270 (F) 800.457.8827

VA/Government

19748 Dearborn Street Chatsworth, CA 91311 (P) 800.966.6662 (F) 800.232.9796

Canadian Office

1000 Clarke Road, Suite 6 London, ON Canada N5V 3A9 (P) 866.546.1151 (F) 519.451.8662